Immediate Itch & Odor Relief!

CastCooler® will...

- RELIEVE ODOR & ITCH
- FRESHEN YOUR CAST
- DRY YOUR CAST

... and may:

- REDUCE SKIN MACERATION
- REDUCE SKIN INFECTION

Moms hurt when their kids hurt.



Now there is something you can do to help your child with the unpleasant experience of wearing a cast. The CastCooler® provides instant

relief from the hot, sweaty, itchy and uncomfortable experience of cast wearing.

CastCooler® keeps casts and splints fresh, itch & odor free.

The CastCooler® will dry a cast which becomes damp from perspiration, humidity, or accidental wetting.

Moisture is the reason that bacterium grows and the cast itches and stinks. The CastCooler® will keep your cast fresh, dry and comfortable by removing moisture from the lining.



CastCooler® is perfect for any splint or cast, including hip-spica casts and is recognized by Int'l Hip Dysplasia Institute.



CastCooler®

Immediate itch & odor relief for your cast, splint or splint cast.







Where can I purchase the CastCooler®? Visit castcooler.com.

We understand cast itch and odor! You are not alone. We know the challenges: itch, odor, and the feeling like you can't escape your increasingly smelly and uncomfortable cast/splint. CastCooler® stops that odor and itch, immediately, with a revolutionary technology that is a breakthrough in cast and splint care. Your life with a cast/splint continues but the CastCooler® will help make your experience itch free, cleaner and dryer.

How often do I use the CastCooler®? Use the CastCooler® as often as you like, it cannot be overused. Whenever your cast or splint is

moist, hot or stinky...use it. No need for painful splint removal or reapplication. Simply apply the CastCooler to the outside of your splint.

How do I use the CastCooler® and for how long? The CastCooler® is so easy to use a child can use it. Simply wrap around your cast, hook to a vacuum and enjoy immediate cooling relief. Use as often as you like for 5-10 minutes daily.

Will the CastCooler® work with any cast or splint? Yes, the CastCooler® is designed to work with all breathable casts and splints: leg, foot, arm, wrist, even hip-spica casts.

Does the CastCooler® require a

special vacuum? No, the CastCooler® is designed for use with any household vacuum. IF however, your cast has become wet or saturated, you MUST use a wet/dry vac as liquid will be quickly removed from the cast lining.

US Patents 7,229,425 & 7,497,838 Additional patents pending.

The CastCooler® benefits you.

With the CastCooler you can have a cast or splint that is:

- Completely Free of itch
- Dry and Comfortable
- Fresh and Clean! (no more stinky smell)

By keeping your cast/splint lining free from moisture, especially moisture from perspiration, the conditions for bacteria growth are removed. Your cast stays clean and fresh. Itch and odor are not issues because you are removing the root cause of each...moisture.

How do I keep the CastCooler® clean? Restore original cling properties by keeping the CastCooler® free from debris by rinsing in clear water as needed and allowing to air dry.



www.castcooler.com